NL

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0481

EN

Instructions for use HeltiQ Honey Wound Dressing

HeltiQ Honey Wound Dressing is a hydro-active, honey-containing dressing for use on small acute and chronic* wounds. HeltiQ Honey Wound Dressing is sterile, contains honey (30%), water and acrylic polymer, and is covered with a polyurethane film. The honey-hydrogel dressing re-hydrates dehydrated tissues and also helps absorb and retain low levels of wound fluid. HeltiQ Honey Wound Dressing helps promote wound healing by maintaining a moist wound environment. The polyurethane film provides the dressing with bacterial barrier properties. The honey wound dressing adheres to the skin well, making it possible to shower and bathe without having to remove the dressing**.

* *Whether it is a chronic wound should be determined by a physician. Use this product for this purpose after consulting the physician.*

** *When placed on greasy skin or moving body parts the dressing might not be waterproof.*

INDICATIONS

HeltiQ Honey Wound Dressing is indicated for small chronic* wounds, such as bedsores; leg ulcers and diabetic foot ulcers. HeltiQ Honey Wound Dressing is also suitable for small acute wounds such as surgical wounds, cuts and abrasions, donor sites of skin grafts, and superficial 1st & 2nd degree burns.

* *Whether it is a chronic wound should be determined by a physician. Use this product for this purpose after consulting the physician.*

CONTRAINDICATIONS

Do not use on individuals with a known sensitivity to the dressing or its components. The dressing should not be used on:

- Wounds that produce significant amounts of wound fluid
- Third degree burns
- Narrow deep wounds or cavities

WARNINGS AND PRECAUTIONS

HeltiQ Honey Wound Dressing may be used on contaminated wounds and wounds with microorganisms. However, if wounds show signs of infections*, treatment with HeltiQ Honey Wound Dressing should be discontinued. Any further treatment for infection should be initiated under medical supervision. The bacterial barrier property of the dressing is compromised if the dressing is punctured or damaged, is not fully adhered to the skin or when wound fluid leaks from under the edges. Care should be taken on removal to avoid skin damage due to the adhesive edges, especially if repeated applications are necessary. Do not use if the packaging is damaged or compromised in any way.

* *Signs of infection include: odor, pain, necrotic tissue, redness of the skin, swelling, wound fluid, and delayed healing.*

EN

Instructions for use HeltiQ Honey Wound Dressing

INSTRUCTIONS FOR USE

1. Inspect and, if necessary, cleanse the wound with water or saline solution.
2. Make sure the patch has sufficient overlap on healthy skin.
3. Remove the sterile dressing from the package.
4. Peel back half of the white plastic protective liner.
5. Gently place the exposed gel side of the patch over the wound, while removing the second half of the white plastic protective liner.
6. Once it is fully in place, remove the top liners from the back of the dressing. Ensure that the adhesive of the HeltiQ Honey Wound Dressing does not come into contact with the wound surface and is properly adhered.
7. Inspect the wound and the surrounding tissue regularly.
8. In the event of an intolerance to the dressing or one of its components, remove the dressing using appropriate techniques and clean the area carefully.

HOW OFTEN SHOULD THE DRESSING BE CHANGED?

The dressing can remain in place for up to five days, depending on the amount of wound fluid. Replace the dressing before it becomes saturated with wound fluid, or if it has been damaged by puncture, partial removal or leakage of wound fluid. Complete and gentle removal of the dressing may be aided by moistening it slightly. Each dressing is for single use. Do not use after the expiration date.

STORAGE PRECAUTIONS

Store between 5 °C and 25 °C.
Avoid direct sunlight and moisture.

Theo Manufacturing B.V.
Sleperweg 44, 6222 NK Maastricht, Nederland

Koninklijke Utermöhlen NV
De Overweg 1, 8471 ZA Wolvega, Nederland
www.heltiq.nl

This is a medical device.