Packungsbeilage
bitte beachten

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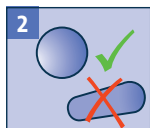
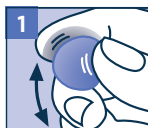
EN OHROPAX® Gute Nacht sleep earplugs are the best way to ensure a peaceful, refreshing sleep.

The earplugs are made from pure, high-quality silicon and are easy to shape, guaranteeing that they will sit perfectly on your ear. Known as the ON-Ear principle, this avoids uncomfortable pressure in your ear and makes the earplugs very comfortable to wear. OHROPAX® Gute Nacht earplugs are reusable, and are scientifically proven to be gentle on the skin. The sound insulation value of SNR 23 dB reduces ambient noise to a level which lowers the alertness of your hearing. So OHROPAX® Gute Nacht earplugs help you enjoy a deep sleep without missing important alert sounds, such as your alarm clock. And you begin your day relaxed and refreshed. OHROPAX® Gute Nacht earplugs are equally well-suited for all sleeping positions.

Reusable silicone earplugs against harmful noise (EN 352-2:2002, EU 2016 / 425)

Instructions for use (see picture): 1. Soften the plug by vigorous kneading. 2. Form it into a ball (no elongated shape!). 3. Seal the opening of the ear canal with gentle pressure.

Do not insert into
the ear canal!

Use entire earplug – do not split!

Removal: Use your fingers to remove the plug in one piece. Do not use any auxiliary means. The plug can be loosened by slightly pushing the back of the ear.

Further information: • Insert the earplugs as described and check that they fit correctly. • Ensure hygienic use. Your hands, ears and earplugs should be clean and dry. • The earplugs should be regularly inspected for usability as well as for impurities. • If they still adhere to your ear and are clean and dry, they can be reused. • Replace soiled or damaged earplugs or such plugs that have become less effective. • Cleaning: Light impurities can be carefully wiped off using a cloth and warm water. Wait until completely air-dried. • Wear earplugs at all times in noisy environments. • Sudden or fast removal of the earplug out of the ear may damage the eardrum. • Failure to comply with these instructions may significantly impair the protective effect of the product. • This product may be adversely affected by certain chemical substances (e.g. hand cream). Further information can be sought from the manufacturer. • Keep away from small children (danger of swallowing). • Can be used by children aged six and older; use only under adult supervision. • Ensure that the plug is large enough so that it cannot accidentally enter the ear canal. • If it cannot be removed using your fingers, please consult a physician.

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OHROPAX GmbH, Am Kappengraben 17, 61273 Wehrheim