
OHROPAX® Kids

EN 352-2:2020, EU 2016/425

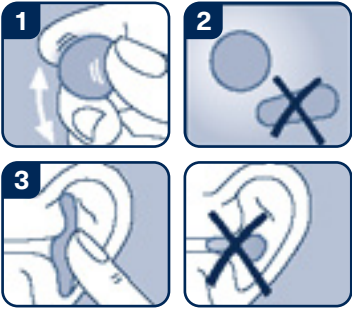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Reusable silicone earplugs against harmful noise

Instructions for use (see picture): 1. Knead the silicone plug thoroughly until the material becomes pleasantly soft. 2. Shape into a ball (no elongated form!). 3. Seal the ear canal tightly by gently pressing the ball. **Do not insert into the ear canal! Use entire earplug – do not split!**

Removal: Use your fingers to remove the plug in one piece. Do not use any auxiliary means! The plug can be loosened by slightly pushing the back of the ear.

Further information: • Insert the earplugs as described and check that they fit correctly. • This product is intended only for people with smaller ear canals. • Ensure hygienic use. Your hands, ears and earplugs should be clean and dry. • The earplugs should be regularly inspected for usability as well as for the necessity of cleaning. • If they still adhere to your ear and are clean and dry, they can be reused. • Cleaning: Light impurities can be carefully wiped off using a cloth and warm water. Wait until completely air-dried. • Wear earplugs at all times in noisy environments. • For water protection: Do not use when diving at depths greater than 1 m. Highly active movements in water, such as jumping or freestyle swimming, may loosen the plugs. • Ask your physician before using earplugs if you have an ear infection, an eardrum injury or a grommet inside the ear. • Failure to comply with these instructions may significantly impair the protective effect of the product. • This product may be adversely affected by certain chemical substances (e.g. hand cream). Further information can be sought from the manufacturer. • Keep away from small children (danger of swallowing). • Can be used by children aged six and older; use only under adult supervision. • Ensure that the earplug is sufficiently large to prevent accidental insertion into the ear canal. • If removal with fingers becomes impossible, seek medical advice.

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- A CHEMOMEDICA GmbH, 1010 Wien, www.chemomedica.at
- P Uriach Lusa, S.A., 1495-061 Miraflores, www.uriach.com
OHROPAX GmbH, Am Kappenberg 17