COMPEED® - Be Unstoppable!

10 x 4.2 x 6.8 cm

2696091

14

55709951

Rev. Date: 06/2018

Laboratoire HRA Pharma
200 avenue de Paris
93220 Châtilion - France
Made in EU

www.compeed.com

0459

CE

INSTRUCTIONS:

1. Clean and dry skin before use, ensuring blister area is free of creams and oils.

2. Remove the bottom paper, avoiding touching the adhesive side.

3. Apply plaster firmly over blister, ensuring the edges are well smoothed down.

4. Do not remove plaster until it starts to detach (Note: may stay in place up to several days).

5. To remove plaster, do not pull upwards but slowly stretch along the skin.

WARNINGS:

- Diabetics: consult with your doctor before use.
- Do not use on infected or inflamed skin.
- Do not reuse, to avoid infection.
- If blister worsens or does not heal stop exercising and consult a doctor.
- COMPEED® plaster may irritate sensitive skin.
- Glue residue on clothing can be removed with white spirit, considering the special care instructions of the fabric.

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55710108

Rev. Date: 06/2018

Laboratoire HRA Pharma
200 avenue de Paris
93220 Châtilion - France
Made in EU

www.compeed.com

0459

CE

COMPEED® hydrocolloid technology is an active gel with moisture absorbing particles. COMPEED® plaster acts like a second skin to support the natural moisture balance, to:

- Relieve blister pain instantly.
- Protect and cushion against rubbing.
- Offer fast wound healing.

Stays in place for several days. Individual experience might vary.

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